



LOCOMOTOR ROTA CUSHION

LOCO-068 Standard
LOCO-078 Large

For more information contact the company you purchased the product from or contact customer services at:

Select...Healthcare (UK) Ltd
Unit 2,
Albert Road
Rushden
Northants
NN10 0BU

Tel: 01933 411 729
Fax: 01933 410 213

INSTRUCTIONS FOR USE

The ROTA CUSHION is designed for seated transfers only. Place the cushion on chair or car seat and sit directly on the cushion. The swivel action of the cushion allows you to rotate smoothly into the correct position on the chair or car seat with minimum effort.

DIMENSIONS:

Diameter: Standard 380mm
 Large 450mm

MATERIALS:

Cotton outer.
Nylon inner.
Steel rotating disc with velcro attachment.

CARE & USE:

Machine wash up to 71 degrees.
Wash coloureds separately.
Before washing remove the steel disc and replace once the fabric cushion has been laundered.

IMPORTANT:

All patient moving and handling tasks should be carried out in accordance with approved patient handling techniques and after a full assessment of the patient, the task and the environment in which the operation is to take place.