



SmitCare

14 Camberley Business Centre

Camberley, Surrey GU15 3DP

Tel: 01276 677888

Email: sales@smitcare.co.uk

www.smitcare.co.uk

SmitCare Product Information

Straight Transfer Board **Codes DTB24S and DTB30S**

Manufactured for SmitCare

Maximum user weight 130kg (20 stone)

DTB24S Length Overall: 610mm (24")

DTB30S Length Overall: 750mm (30")

Width: 210 mm (8 ¼")



The SmitCare Straight Transfer Board is a moving and handling device used by people with limited mobility to transfer from one sitting position to another. The type and extent of mobility impairment may dictate the best method of use for the board, so it is advisable to consult with a trained Health Professional before first use, such as an Occupational Therapist or Moving and Handling Advisor.

The following points should be noted when using the Transfer Board:

- 1) Make sure the surfaces you are transferring between are as close as possible and ideally at a similar height.
- 2) Make sure that the Transfer Board is the correct way up, with the chamfered edges on top.
- 3) When transferring to or from a mobile surface, such as a wheelchair or glideabout, ensure that the brakes are applied.

Clean regularly with a solution of mild detergent in warm water applied with a damp sponge. Dry thoroughly. Do not clean with harsh abrasives.

Questions / comments? Please contact our Customer Services department
on: **01276 677888**