



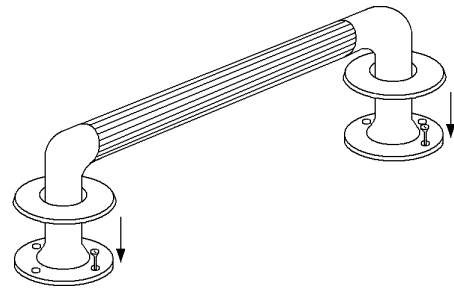

SmitCare
14 Camberley Business Centre
Camberley, GU15 3DP
Tel: +44 (0)1276 677888
Email:sales@smitcare.co.uk
www.smitcare.co.uk

Aquarian Plastic Fluted Grab Rails

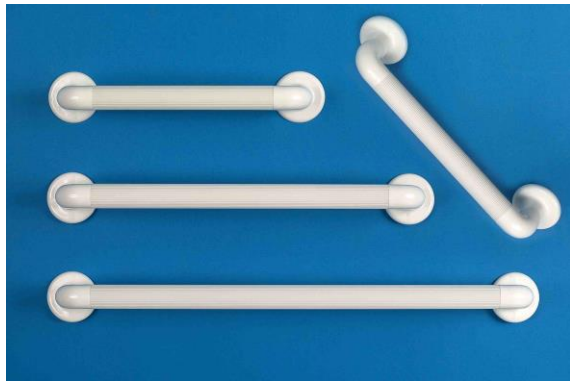

AQGR12 12" (305mm) Aquarian Fluted Grab Rail

AQGR18 18" (460mm) Aquarian Fluted Grab Rail

AQGR24 24" (610mm) Aquarian Fluted Grab Rail

Manufactured for SmitCare

SmitCare Aquarian Fluted Grab Rails are intended to provide support around the home. Each flange has five holes, three of which must be used to secure the rail. The holes are intended to take No.10 (5mm) stainless steel countersunk screws. The Grab Rails have screw covers that clip over the flanges for a neat appearance. The tube is manufactured from extruded UV resistant PVC and the ends are injection moulded ABS.

Important Note:

Grab Rails should be fitted by a competent person with appropriate fittings for the material of the wall.

Weight Rating:

The Grab Rails have been tested for a load of 190kg (30 stone)