


User Instructions

Clip & Stack Overview

The Clip & Stack is designed to raise furniture by a maximum of 3½" (88mm), using a simple system of clip-together stacks to achieve the required height.

After raising, furniture must still be supported at all designed support points. If the furniture has more than four support points, more than one set of raisers will be required.

There are three main components to the system; the Disc that secures to the furniture (Fig 1), the Clip-on height adjusting section (Fig 2) and the push on soft adaptor (Fig 3) for use on hard floors.

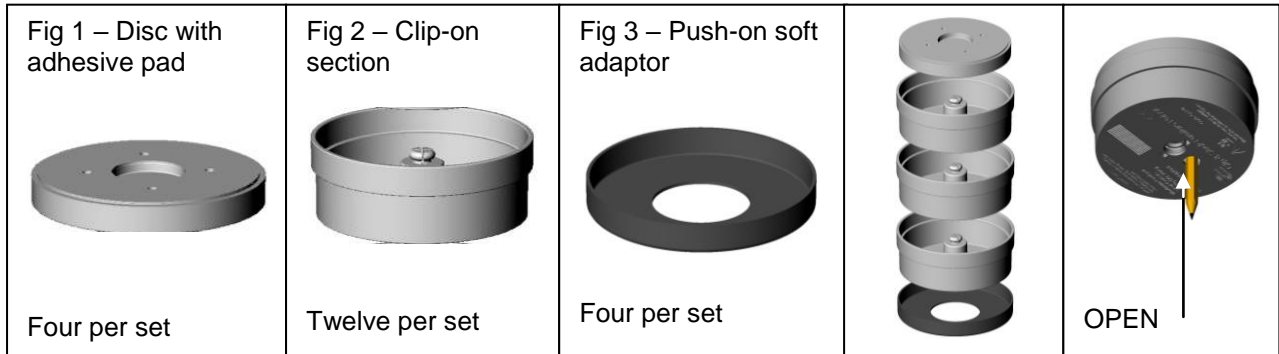

Separating Clip-On Sections and Discs

The Clip-on sections pull apart. Prior to fitting, remove discs from Clip-ons by pushing each Disc out with a suitable tool, such as a screwdriver, inserted through the offset hole marked OPEN in the underside of the Clip-on.

Fitting Discs to Furniture

Elevate the furniture and support it safely during fitting of the raisers. Do not elevate occupied furniture. Attach the Discs to the support points on the furniture first. To attach Disc to furniture, ensure that the attachment/support point on the furniture is clean and free from dust and oil etc. Peel off the protective paper from the top of the adhesive pad and place the Disc under the support point of the furniture. Repeat this procedure for the other Discs. To maintain adhesion, avoid tilting the furniture excessively during, or after, fitting the raisers. If there is doubt about the security of the adhesion of the Disc, it may be screwed to the furniture with two or more screws as required.

Adding Clip-On Sections

Once the Discs are fitted to the furniture, a raise of ½" (13mm) will have been achieved. Each Clip-on section will add an additional 1" (25mm) to the height raised. If your furniture is situated on hard surface flooring, such as wood or laminate, the push on soft adaptors may be added to the bottom sections to avoid floor damage and prevent the furniture moving. If the furniture is situated on carpet then the push on soft adaptors may not be required.

Note

- In any situation where furniture is likely to be regularly moved, we recommend use of the MLA1/U Link Arms with the Clip & Stack Raisers
- Do not move occupied furniture
- Do not raise beyond specification (88mm)

Specification

Maximum Load per set of four raisers 78 stone (500Kg)
 Possible furniture raise: ½"(13mm), 1½"(38mm), 2½"(63mm), 3½"(88mm)
 Suitable for furniture with legs 2.6"(65mm) diameter, 1.5"(45mm) square or larger
 Warranted to specification for two years
 Pat. No: GB0819619