GB

1. Turn knobs anti-clockwise until brackets are fully extended. Note: Brackets for the 15-cm Super & Deluxe model will require fitting first.
2. Lift existing seat and place raised toilet seat on to WC.
3. Turn knobs clockwise until rubber buffers touch WC. Do not overighten. Test seat to ensure it is securely fitted.
4. Check tightness of fit from time to time.
5. Clean with non-scratch detergent.
6. Thermal disinfection at 85° C for one minute.

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# Welcome To Gordon Ellis & Co. Home Healthcare Products

## 3 Year Guarantee

The **ASHBY RAISED TOILET SEAT** is manufactured by Gordon Ellis & Co. to the highest of standards. We warrant to the end-user customer that all the components of the **ASHBY** seat will be free from any defects in materials and workmanship, and will perform effectively under normal use conditions for a period of three years from date of purchase.

Gordon Ellis & Co. will repair or replace, as applicable, any component covered by this warranty, which is found to be defective. This does not affect your statutory rights.

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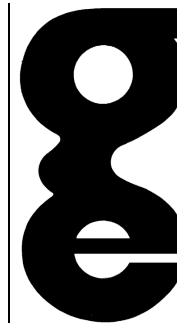
For other details of Gordon Ellis products for elderly and disabled people, please contact your supplier, or Gordon Ellis direct.

[www.gordonellis.com](http://www.gordonellis.com)

**GORDON  
ELLIS  
& CO**

**MANUFACTURERS IN  
WOOD AND PLASTIC  
ESTABLISHED 1893**

**Ashby Raised  
Toilet Seat  
Super / Deluxe**

TRENT LANE  
CASTLE DONINGTON  
DERBY DE74 2NP  
ENGLAND  
TEL: (01332) 810504  
INT: + 44 1332 810504  
FAX: (01332) 850830  
INT: + 44 1332 850830  
gordon.ellis@gordonellis.co.uk  
www.gordonellis.com

Company registered in England:  
No. 231734  
VAT No. GB 428 4666 26

6212/0301