



LOCOMOTOR SLIDE SHEET

Regular: **LOCO-001 Standard**
 LOCO-002 Long
 LOCO-003 Wide

Ultra: **LOCO-001X Standard**
 LOCO-002X Long
 LOCO-003X Wide
 LOCO-012 Compact
 LOCO-013 Maxi
 LOCO-067 Maxi Plus
 LOCO-076 Mini
 LOCO-115 Extra Long

For more information contact the company you purchased the product from or contact customer services at:

Select Healthcare (UK) Ltd
Unit 2
Albert Road
Rushden
Northants
NN10 0BU

Tel: 01933 411 729
Fax: 01933 410 213

The LOCOMOTOR SLIDE SHEET is a fabric cylinder with a low friction inner surface allowing an individual to be repositioned without the need to lift. The LOCOMOTOR SLIDE SHEET is available in various sizes and is commonly used for the following patient handling tasks: -

- 1) Patient turning.
- 2) Moving a patient up the bed to a sitting position.

The LOCOMOTOR SLIDE SHEET can also be used for a variety of repositioning tasks. The multidirectional glide of the fabric allows patient's to be moved in any direction on the bed and reduces shearing to the patient's body.

INSTRUCTIONS FOR USE

1) **Patient turning.**

Roll the patient onto the LOCOMOTOR SLIDE SHEET ensuring that the hip and shoulder are over the slide sheet.

To turn the patient, take hold of the top layer of the slide sheet close to the patient's hip and shoulder and pull gently outwards and upwards. The patient will turn onto his/her side.

Alternatively, slide the patient across the bed first and then roll the patient back into the middle of the bed.

To remove the sheet, take hold of the bottom layer of the sheet and gently slide out from beneath the patient.

2) Moving a patient up the bed.

Roll the patient onto the LOCOMOTOR SLIDE SHEET. If the patient can maintain a sitting position then the slide sheet should be placed under the hip and heel (if required). If the patient cannot sit then the sheet should be positioned under the shoulder and hip. Then using an approved manual handling technique, slide the patient up the bed.

IMPORTANT

All patient moving and handling tasks should be carried out in accordance with approved patient handling techniques and after a full assessment of the patient, the task and the environment in which the operation is to take place.

SPECIFICATION

Ultra: Silicone coated nylon.

Regular: Nylon

DIMENSIONS FOR REGUALR & ULTRA SLIDE SHEETS

Standard: 1200mm x 710mm
Long: 1450mm x 710mm
Wide: 1200mm x 1000mm
Compact: 720mm x 700mm
Maxi: 1950mm x 700mm
Maxi Plus: 1950mm x 1400mm
Mini: 600mm x 450mm
Extra Long: 2000mm x 710mm

CARE & USE

May be washed at 71°C if required or autoclaved.

May be sterilised using Sodium Dichloroso-Cyanurate or Sodium Hypochlorite – wipe only the rinse with water.

Regularly check for wear and do not use if there is any sign of deterioration.