

23.02.05

THE LOCOMOTOR TURN-TABLE

LOCO-034 300mm

LOCO-035 380mm

SELECT... HEALTH CARE (UK) LTD.

2 ALBERT ROAD

RUSHDEN

NORTHANTS

NN10 OBU

TEL: (01933) 411729

FAX: (01933) 410213

INSTRUCTIONS FOR USE

1. Standing Transfer

The TURN-TABLE can be used either independently by the client or with assistance by one or two nursing/care staff.

1.1 Independent Transfer

The client places the TURN-TABLE on the floor in front of their seated position. The client then places both feet on the TURN-TABLE and transfers to an alternative seated position such as a wheelchair/bed or commode, using a transfer board or their own upper body strength to complete their manoeuvre.

1.2 Assisted Transfer

As above however the client is helped to a standing position using an approved manual handling technique and is rotated on the TURN-TABLE by one or both of the carers to an alternative seated position.

2. Seated Transfer

The soft surface and flexible design of the TURN-TABLE allows clients to sit on the TURN-TABLE and rotate themselves either on a bed or chair as well as in and out of a car seat.

IMPORTANT

ALL PATIENT HANDLING OPERATIONS SHOULD BE CARRIED OUT IN ACCORDANCE WITH APPROVED MANUAL HANDLING TECHNIQUES AND AFTER A FULL ASSESSMENT OF THE PATIENT AND THE ENVIRONMENT IN WHICH THE OPERATION IS TO TAKE PLACE.

DIMENSIONS

LOCO-034/035	Depth	8mm
	Diameter	300mm/380mm
	Safe working load	127kg

CARE AND USE

Be vigilant check device integrity before use.

Upper surface of the TURN-TABLE may be washed with warm soapy water or brushed with a stiff brush to remove dirt.

The base of the TURN-TABLE may be washed with warm soapy water.

NOTE

Always ensure that the carer/nurse anchors the turntable with their foot whilst carrying out the transfer to prevent any lateral movement of the disc.

