


# LOCO-102 LOCOMOTOR ROTA STAND

ENSURE THESE INSTRUCTIONS ARE READ AND FULLY UNDERSTOOD BEFORE USE.  
PLEASE KEEP IN A SAFE PLACE FOR FUTURE REFERENCE.

HEALTH AND SOCIAL SERVICES -  
THESE INSTRUCTIONS SHOULD BE LEFT WITH THE USER OF THE LOCOMOTOR ROTA  
STAND TO CONSULT AS REQUIRED.

The ROTA STAND is packed in a strong purpose built carton, to ensure safe delivery.  
Please retain the carton if possible. Inside this carton you will find the ROTA STAND and

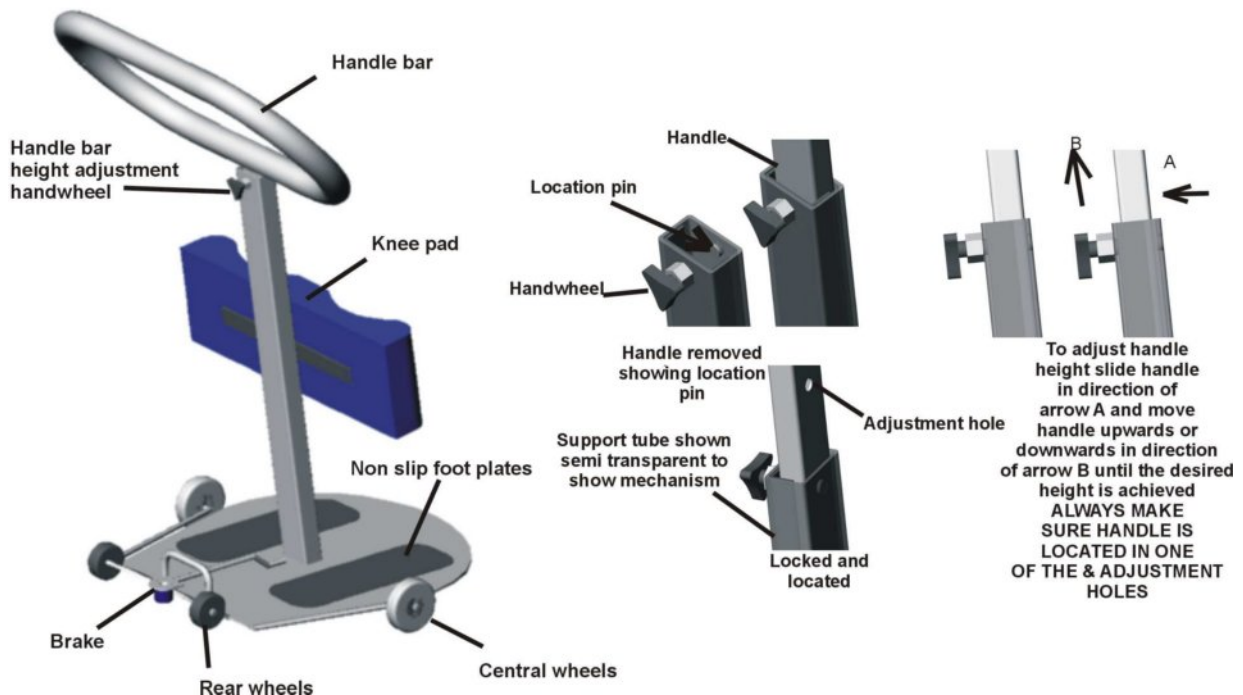

## Instructions

### 1. Getting Started with your Locomotor ROTA STAND

- 1.1. Remove all items from the carton.
- 1.2. Identify the items against the packing list.

### 2. Assembly of your Locomotor ROTA STAND (See diagram)

- 2.1. The ROTA STAND comes fully assembled and ready to use

### 3. Using the Locomotor ROTA STAND .

- 3.1. The ROTA STAND has been designed to allow staff to transfer a patient/client from one seated position to another. Before a transfer can take place staff should satisfy themselves through an appropriate risk assessment that the patient/client is able to bear weight independently

# LOCO-102 LOCOMOTOR ROTA STAND

- 3.2. Before using your **ROTA STAND** adjust the height of the handlebar to suit the patient/client. The **ROTA STAND** can be manoeuvred on its rear wheels and brought to position in front of the patient/client. Assist the patient/client to the front of their chair, ready to stand and place the patient/client's feet on the nonslip foot plate and gently wheel the **ROTA STAND** up to the patient/client on its central wheels using the foot bar provided. The **ROTA STAND** should end up with the patient/client's shins firmly against the shin pad. From this position put your foot on the brake in order to stabilize the **ROTA STAND** and encourage the patient/client, using the handlebar, to pull themselves into a standing position. If more assistance is required then a second nurse/carer may assist the patient/client to stand using a handling belt around their waist.
- 3.3. Once in a standing position, take your foot off the brake and gently rotate the **ROTA STAND** to the position required and encourage the patient/client to sit, reversing the above procedure.

**NB ALL PATIENT HANDLING TASKS SHOULD BE CARRIED OUT AFTER A FULL RISK ASSESSMENT OF THE PATIENT AND THE ENVIRONMENT IN WHICH THE TASK IS TO BE CARRIED OUT.**

## 4. Cleaning your Locomotor ROTA STAND

- 4.1. Only use a non-abrasive cleaner or disinfectant on the surfaces on the Locomotor **ROTA STAND** . i.e. Isopropyl alcohol at 70% concentration.

## 5. Some Do's and Don'ts.

- 5.1. **DO** ensure that you have read these instructions and understood them fully, including the glossary of symbols in the bottom of these instructions
- 5.2. **DO** ensure that you carry out an inspection of the Locomotor **ROTA STAND** on a regular basis.
- 5.3. **DON'T** let children play with the Locomotor **ROTA STAND** .
- 5.4. **DON'T** exceed the safe working load of a 160kgs person (25 stones).

## TECHNICAL SPECIFICATION

<b>SAFE WORKING LOAD</b>	<b>160kg</b>
<b>OVERALL LENGTH</b>	<b>500mm</b>
<b>WIDTH</b>	<b>520mm</b>
<b>HEIGHT</b>	<b>Min 800mm Max 2040mm</b>
<b>WEIGHT</b>	<b>14kg</b>
<b>CONSTRUCTION</b>	<b>Nylon coated stainless steel &amp; mild steel</b>

## GLOSSARY



Attention: Consult accompanying documents before use.