



LOCOMOTOR QUILTED FLAT UNI-SLIDE CHAIR

LOCO-192

For more information contact the company you purchased the product from or contact customer services at:

Select Healthcare (UK) Ltd
Unit 2
Albert Road
Rushden
Northants
NN10 0BU

Tel: 01933 411 729
Fax: 01933 410 213

INSTRUCTIONS FOR USE

Fold the UNI-SLIDE in half and place on the chair so that the non-slip rubber material is face down on the chair and the label with the arrow is face up and pointing towards the back of the chair.

Sit the patient on the UNI-SLIDE as far back in the chair as possible.

The construction of the UNI-SLIDE will prevent the patient from sliding down in the chair.

The handholds allow nurses/ carers a positive grip should the need arise to slide a patient back in the chair.

DIMENSIONS:

90cm x 45cm

MATERIALS:

Quilted polyester cotton.
Nylon.
Webbing

CARE & USE:

Machine wash up to 71°C.
Wash coloureds separately.

IMPORTANT:

All patient moving and handling tasks should be carried out in accordance with approved patient handling techniques and after a full assessment of the patient, the task and the environment in which the operation is to take place.