



## **LOCOMOTOR QUILTED FLAT UNI-SLIDE BED**

### **LOCO-193**

For more information contact the company you purchased the product from or contact customer services at:

Select Healthcare (UK) Ltd  
Unit 2  
Albert Road  
Rushden  
Northants  
NN10 0BU

Tel: 01933 411 729  
Fax: 01933 410 213

### **INSTRUCTIONS FOR USE**

Fold the UNI-SLIDE in half and place in the desired position, ensuring the arrow is pointing towards the head of the bed.

Hoist the patient into bed or use a slide sheet to position the patient on the bed.

If the patient needs re-positioning up the bed, using an approved technique, use the handholds to slide the patient up the bed.

The construction of the uni-slide will prevent the patient sliding down the bed.

### **DIMENSIONS:**

160cm x 70cm

### **MATERIALS:**

Quilted Polyester cotton.  
Nylon.  
Webbing.

### **CARE & USE:**

Machine wash up to 71°C.  
Wash coloureds separately.

### **IMPORTANT:**

All patient moving and handling tasks should be carried out in accordance with approved patient handling techniques and after a full assessment of the patient, the task and the environment in which the operation is to take place.