User Instructions To be left with the user.

The LOCOMOTOR DUO TRANSFER BOARDS are designed to allow non or semi weight-bearing patients to transfer independently or to be assisted by trained members of staff from one seated position to another with or without additional equipment. The LOCOMOTOR DUO TRANSFER BOARDS provide a smooth low friction surface that bridges the gap between two surfaces. For example bed or chair to wheel chair

IMPORTANT:

ALL PATIENT HANDLING TASKS SHOULD BE CARRIED OUT IN ACCORDANCE WITH APPROVED MANUAL HANDLING TECHNIQUES AND AFTER A FULL RISK ASSSESSMENT.

USE of the LOCOMOTOR DUO TRANSFER BOARDS

Before every use, check that the board is clean and free from cracks or damage. Always keep your LOCOMOTOR DUO TRANSFER BOARD clean.

POSITIONING:

- ALWAYS ENSURE THAT THE NON-SLIP STRIPS ARE FACING DOWNWARDS AND ARE IN MAXIMUM CONTACT WITH THE SURFACES BETWEEN WHICH THE PATIENT IS TRANSFERRING AND THAT THESE SURFACES ARE AS CLOSE TOGETHER AS POSSIBLE.
- ENSURE SURFACES HAVE NO RISK OF MOVING DURING THE TRANSFER AND ARE POSITIONED TO FACILITATE A 'SIDE TRANSFER'.
- ORIENTATE THE BOARD WITH ITS STRAIGHT SIDE TOWARDS THE PATIENT
- THE TRANSFER SHOULD BE MADE WITH THE PATIENT MOVING 'SIDE WAYS' IN A SEATED POSITION SLIDING ALONG THE LENGTH OF THE BOARD.

Commonly the LOCOMOTOR DUO TRANSFER BOARD is positioned by sliding it beneath one hip whilst the patient is in a seated position. Then either independently or with assistance from trained staff the patient slides the length of the LOCOMOTOR DUO TRANSFER BOARD to the required seated position. The LOCOMOTOR DUO TRANSFER BOARD is then removed by tilting the patient and sliding it from beneath the patient.


SPECIFICATION

Maximum patient weight 150 Kg. Dimensions 75cm long, 22cm wide. Weight 1420 gm. LOCO-401B has no handles LOCO-400B has handles to assist the patients movement Warranted to specification for 1 year. Do not store in direct sunlight. Wash with a proprietary detergent. Disinfect using a proprietary disinfectant. Do not use abrasive cleaning materials or substance Can be pressure washed at 79C. For technical help contact:email: info@selecthealthcare.co.uk Or Tel: (01933) 411729 Fax: (01933) 410213

SPARES

LOCO-402 SPARE SET OF HANDLES



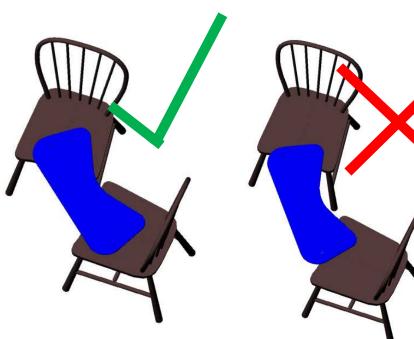

Manufactured and distributed by:

Select Healthcare (UK) Ltd. Unit 2, Albert Road, Rushden, Northants, NN10 0BU. Tel: (01933) 411729 Fax: (01933) 410213 email: info@selecthealthcare.co.uk website: www.selecthealthcare.co.uk



