

ULTRA Swivel Bather™

Swivel Bather is a registered trade mark of Myco Limited

User Instructions



ULTRA Swivel Bather Overview

- The width of the Ultra Swivel Bather base frame can be adjusted from 26" (660mm) to 30" (760mm).
- The Ultra Swivel Bather is supplied with the Locking Lever on the right hand side of the frame. The lever can be changed to the left hand side of the frame by removing the pivot bolt, changing the side of the lever, then reinserting the bolt in the appropriate place.
- The Ultra Swivel Bather should be cleaned regularly with a solution of mild detergent and household disinfectant in warm water. Do not use abrasive cleaning products.

Maximum user weight 25 stone (160Kg)

Adjusting the width of the base frame

Adjust the length of each arm of the base frame by turning the grey end covers:

- turn clockwise to shorten arm
- turn anti-clockwise to lengthen the arm.



Turn Clockwise to shorten arm




Turn Anti-clockwise to lengthen arm

Fitting the base frame to the bath

- Rest the base frame on the bath.
- At least 1 ¼" (30mm) of each arm must rest on the side of the bath.
- Adjust the base width as necessary to ensure a secure fitting.
- The rivets at the end of the grey end covers must be turned to the side, so they do not rest on the bath.
- Turn all four positioning screws until the black rubber ends are just touching the sides of the bath. Do not over tighten.

Safety Note: If the Swivel Bather is removed and replaced over the bath, it is important to ensure that the positioning screws are readjusted as detailed above.

Adjustable Grey PVC Ends

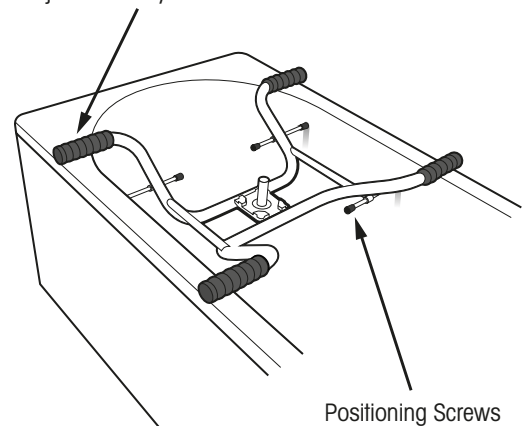


Positioning Screws

Fitting the seat

- Position the Locking Lever so that it is on the right hand side of the seat (unless changed to left hand side - see paragraph 1).
- The socket under the seat locates over the swivel pin of the base frame. Press down on the seat so that the seat unit is fully over the swivel pin and touching the base. Ensure that the seat rotates freely.
- The seat will lock at each 90° interval as the locking lever drops into position. Lift the Locking Lever to swivel the seat.

Locking Lever

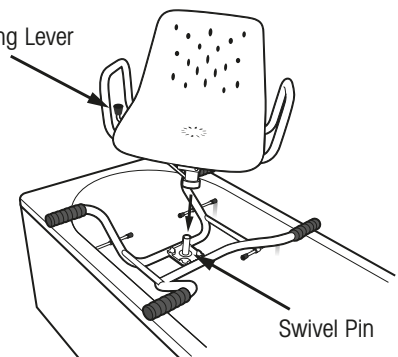

Swivel Pin

Using the Swivel Bather

- Swivel the seat to face the side of the bath and lock into position. The user can now be seated.
- Unlock the seat by lifting the locking lever and transfer legs over the side of the bath.
- Lock the seat when over the bath for extra security while washing or showering

Manufacturer:

SmitCare, 14 Camberley Business Centre, Camberley, Surrey, GU15 3DP

E-mail: sales@smitcare.co.uk Website: www.smitcare.co.uk