

Instructions for the assembly and safe use of the Apex Mobile Commode

Your Apex Mobile Commode has been designed for comfort and ease of use.

The box contains the following

Frame with four braked castors
 Back rest
 Two removable arms
 Pair footrests (adjustable height)
 Black seat cushion
 Pot with lid
 Pair of handwheel bolts
 Code SAC360

Weight limit 190kg (30st)

Manufactured in China for SmitCare

Actual product may differ slightly from that shown



Overview

The Apex Mobile Commode is comfortable and robust, being primarily constructed from chromed steel. The removable backrest, arms and footrest ensure that it can be adapted to suit the requirements of most users, while the four braked castors enable ease of movement and security.

It is important to consider the suitability of this product for the user and to ensure that any potential risks are considered.

- 1) To assemble your Apex Mobile Commode place the frame on the floor, remove the cardboard protection tube from between the back wheels and remove the protective cellophane from the tyres (if applicable). Secure the four brakes by applying downward pressure to each brake lever.
- 2) Remove the black plastic "mushrooms" from the frame (if applicable) to reveal the holes to receive the backrest. Insert the backrest fully, and then screw in the handwheel bolts to secure the backrest in position.
- 3) Slide the pot into the receiving channels from the rear of the seat, then position the padded cushion on top of the seat
- 4) Fit the arms into the tubular slots at the sides of the frame and secure with the sprung pin that locates in the hole in the armrest tube.
- 5) Hook the footrests onto the front of the commode frame, then rotate them into position until they lock in place. The footplates can be adjusted in height by the pinch bolt.

NOTE: Get advice as to the best way to lift or transfer users on to the commode. Think of your back. Remove pot before pushing the commode back over the toilet

Cleaning: Clean with a solution of mild detergent and household disinfectant in warm water. Dry thoroughly. It is not recommended that bleach should be used on the metalwork. However, from time to time this may be necessary, in which case wipe off any surplus liquid as soon as possible

Questions / comments? Please phone
 Customer Services on: **01276 677888**