

Instructions for the assembly and safe use of the Adjustable Height Mobile Commode

The box contains the following

Frame
4 x Extension leg with braked castor
Back rest
Two removable arms
Pair footrests (adjustable height)
Black seat cushion
Pot with lid
Pair of handwheel bolts
Code: SAC360A

Weight limit 190kg (30st)

Manufactured in China for SmitCare

Actual product may differ slightly from that shown



Overview

The Adjustable Height Mobile Commode is comfortable and robust, being constructed from powder coated steel. The removable backrest, arms, footrests and adjustable height legs ensure that it can be adapted to suit the requirements of users, while the braked castors enable ease of movement.

It is important to consider the suitability of this product for the user and to ensure that any potential risks are considered.

- 1) To assemble your Mobile Commode place the frame on the floor, remove the cardboard protection tube from between the back wheels and remove the protective cellophane from the tyres (if applicable).
- 2) Remove the four E-clips from the frame so the extension legs can be inserted and ensure they are all at the same height setting.
- 3) Remove the black plastic "mushrooms" from the frame (if applicable) to reveal the holes to receive the backrest. Insert the backrest and screw in the handwheel bolts to secure in position.
- 4) Slide the pot into the receiving channels from the rear of the seat, then position the padded cushion on top of the seat
- 5) Fit the arms into the tubular slots at the sides of the frame and secure with the sprung pin.
- 6) Hook the footrests onto the front of the commode frame, then rotate them into position until they lock in place. The footplates can be adjusted in height by the pinch bolt.

NOTE: Get advice as to the best way to lift or transfer users on to the commode.
Remove pot before pushing the commode back over the toilet.

Cleaning: Clean with a solution of mild detergent and household disinfectant in warm water and dry thoroughly. Not suitable for machine cleaning.

Questions / comments? Please phone
Customer Services on: **01276 677888**