



SmitCare Limited
14 Camberley Business Centre
Camberley, Surrey GU15 3DP
Tel: 01276 677888
sales@smitcare.co.uk
www.smitcare.co.uk

Instructions for the safe use of a Boomerang Board **Code STBTB**

Manufactured for SmitCare
Maximum user weight 190kg (30 stone)
Length Overall: 740mm (29")
Width: 260 mm (10")



The SmitCare Boomerang Board is a moving and handling device used to transfer from one sitting position to another. The type and extent of mobility impairment may dictate the best method of use for the board, so it is advisable to consult with a health professional before first use, such as a Moving & Handling Advisor or Occupational Therapist.

The following points should be noted when using the Boomerang Transfer Board:

- 1) Make sure the surfaces you are transferring between are as close as possible and ideally at a similar height.
At least 10cm of each end of the transfer board must be supported.
- 2) Make sure that the Boomerang Board is the correct way up. The slip-resistant rubber pads must be on the underside to prevent movement of the board during the transfer.
- 3) When transferring to or from a mobile surface, such as a wheelchair or glideabout, ensure the brakes are applied.
- 4) If possible, having achieved the transfer, place the Boomerang Board in a position where it can be easily reached so that the return transfer can be affected safely.

Questions / comments? Please contact our Customer Services department
on: **01276 677888**