



14 Camberley Business Centre
Camberley, GU15 3DP
Tel: +44 (0)1276 677888
Email:sales@smitcare.co.uk

Instructions for the SmitCare Ultra Bath Step

Important Note

A risk assessment must be carried out for the suitability of this product with the user.

Weight Limit: 440kg (70st)

Step Size: 45x35cm (18 x14")

Product Code: SUBS

Manufactured in China for SmitCare



The SmitCare Ultra Bath Step comprises of four sections that are pressed together to make a 10cm (4") Bath Step. The sections can be separated to make a lower step, or additional sections can be added up to a maximum height of 20cm (8").

The top surface of each section is textured to give a slip-resistant footing, while the bottom section has four rubber feet to provide secure location on the bathroom floor.

The Ultra Bath Step is 45cm x 35cm (18"x14"), which provides a good, stable step for getting in or out of the bath. It is advisable that the height of the step should be approximately the same height as the bottom of the bath, so that it is easier to maintain balance and feel confident stepping over the side of the bath.

Directions for use

1. Adjust the height of the step, referring to the guideline above.
2. Ensure the sections are firmly pressed together.
3. Position step against the side of bath.
4. Test for stability (an uneven floor may be a factor, especially with tiles).
5. Ensure the step is dry, so that it offers maximum grip.
6. Stand centrally on the step with both feet.
7. Step directly into the bath.

Cleaning

To maintain the appearance and effectiveness of the Ultra Bath Step, clean with a non-abrasive anti-bacterial detergent. Dry after cleaning

Questions / comments?

Please contact our Customer Services department on 01276 677888